



DiVa Settings Cheat Sheet

Pre-Menopausal

Treatment/ Pass	1470 (Non-Ablative)			2940 (Ablative)		
	Range of Treatment	Depth (µm)	Density	Range of Treatment	Depth (µm)	Density
Treatment 1/ Pass 1	360°	600	15%			
Pass 2	360°	600	15%	360°	350	14%
Treatment 2/ Pass 1	360°	700	18%			
Pass 2	360°	650	16%	360°	400	14%
Treatment 3/ Pass 1	360°	700	18%			
Pass 2	360°	700	18%	360°	450	14%

DiVa Labial

Treatment/ Pass	1470 (Non-Ablative)			2940 (Ablative)		
	Range of Treatment	Depth (µm)	Density	Range of Treatment	Depth (µm)	Density
Treatment 1	90°	300	7%	90°	100	14%

Use the same settings for treatments 2 & 3.

DiVa Tyte (SkinTyte II)

Intensity	Time (seconds)	Temperature	Notes
15	15	30°	Get to target temperature. Hold temperature for 10-15 minutes per slide.

Post Menopausal

Treatment/ Pass	1470 (Non-Ablative)			2940 (Ablative)		
	Range of Treatment	Depth (µm)	Density	Range of Treatment	Depth (µm)	Density
Treatment 1/ Pass 1	360°	500	6%			
Pass 2	360°	400	4%	360°	200	7%
Treatment 2/ Pass 1	360°	600	15%			
Pass 2	360°	500	12%	360°	250	14%
Treatment 3/ Pass 1	360°	700	18%			
Pass 2	360°	600	15%	360°	300	14%
Treatment 4/ Pass 1	360°	700	18%			
Pass 2	360°	700	18%	360°	350-400	14%

